Fitting instructions

- Thread top elastic strap through top buckles. Repeat for bottom strap and buckles. Place the bottom elastic strap around the head just below the ears. Untwist the strap.
- Place the top elastic strap around the head, above the ears. Untwist the strap.
- Adjust the tension by pulling tabs of each strap, as shown.
- Using both hands, mould the metal nosepiece comfortably to the shape of your nose. Pinching the nosepiece using only one hand may result in less effective respirator performance.
- Strap tension may be decreased without removing the respirator from the head by pushing out on the back of the buckles.
- The seal of the respirator on the face should be checked prior to wearing in the work area.
 - Cover the front of the respirator with both hands, being careful not to disturb the position of the respirator.
 - Inhale sharply. A negative pressure should be felt inside the respirator. If any leakage is detected, adjust the position of the respirator and/or tension of the straps. Retest the seal. Repeat the procedure until the respirator is sealed properly. If you CANNOT achieve a proper fit DO NOT enter the contaminated area. See your supervisor.

Note: do not use with beards or other facial hair that may inhibit contact between the face and the edge of the respirator.

Warning

- As with the use of any respiratory device, the wearer must first be trained in the proper use of the product.
- This product does not protect the wearer against gases, vapours, solvents from paint spray operations or in atmospheres containing less than 19.5% oxygen.
- Use only in adequately ventilated areas containing sufficient oxygen to support life.
- Do not use when concentrations of contaminants are immediately dangerous to life or health.
- Leave the area immediately if:
 - breathing becomes difficult;
 - dizziness or other distress occurs.
- Discard and replace respirator if it becomes damaged or breathing resistance becomes excessive.
- Never alter or modify this device.
- Failure to follow all instructions and warnings on the use of this respirator during all times of exposure can reduce the respirator effectiveness and may result in sickness or permanent disability.
- Cleaning and disinfection. If the 8835 is to be used for more than one shift it should be cleaned at the end of each shift and stored between shifts in a sealed container or bag. To clean the 8835, the 3M face seal cleaner 105 should be used to wipe the face seal gasket of the product.

Approvals

This product has been shown to meet the Basic Safety Requirements under art. 10 and 11B of the European Community Directive 89/686 and is thus CE marked. The product has been examined at the design stage by Berufsgenossenschaftliches Institut für Arbeitssicherheit, Postf. 2043, Alte Heerstr. 111, D-53757 St. Augustin, Germany (0121).

3M United Kingdom PLC.
3M House, P.O. Box 1, Market Place
Bracknell, Berks. RG12 1JU
Tel: 0800 212490

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