3M United Kingdom P.L.C.
3M House, P.O. Box 1, Market Place
Bracknell, Berks. RG12 1JU
Tel: 0870 60 800 60

3M Ireland Ltd.
3M House, Adelphi Centre
Upper Georges Street
Dun Laoghaire, Co. Dublin
Tel: 1800 320 500

3M (East) AG.
Grundstrasse 14, 6343 Rotkreuz
Tel: 041-7994040

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3M (East) AG
Representation Office
Bužanova 2 - HR-10000 Zagreb
Tel: 1/2361-500 - Fax: 1/2302-471

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Print = 90 %
Real size: 210 x 420 mm

3M**Fitting Instructions**

1. First pre-stretch each strap by pulling between both hands at 1-2 inch intervals around the entire length of the strap. (GB) (IRL)
2. Cup the respirator in your hand with the nose-piece at your fingertips, allowing the headbands to hang freely below your hand.
3. Position the respirator under your chin with the nose-piece up.
4. Locate the lower strap below the ears and the upper strap across the crown of the head.
5. Using both hands, mould noseclip to the shape of the lower part of the nose. DO NOT pinch the nose-piece using only one hand as this may result in less effective respirator performance.
6. The seal of the respirator on the face should be fit-checked prior to wearing in the work area.

To check the fit of an unvalved respirator:

- a) Cover the front of the respirator with both hands, being careful not to disturb the position of the respirator.
- b) Exhale sharply. If any air flows around the nose, re-adjust the nose-piece as described in step 5. If air leaks at the respirator edges work the straps back along the sides of the head. If you CANNOT achieve a proper fit DO NOT enter the contaminated area. See your supervisor.

Note: do not use with beards or other facial hair that may inhibit contact between the face and the edge of the respirator.

Warning

1. As with the use of any respiratory device, the wearer must first be trained in the proper use of the product.
2. This product does not protect the wearer against gases, vapours, solvents from paint spray operations or in atmospheres containing less than 19.5% oxygen. (3M definition).
3. Use only in adequately ventilated areas containing sufficient oxygen to support life.
4. Do not use when concentrations of contaminants are immediately dangerous to life or health.
5. Leave the area immediately if:
 - a) breathing becomes difficult
 - b) dizziness or other distress occurs
6. Discard and replace the respirator if it becomes damaged, breathing resistance becomes excessive or at the end of the shift.
7. Never alter or modify this device.
8. Failure to follow all instructions and warnings on the use of this respirator during all times of exposure can reduce the respirator effectiveness and may result in sickness or permanent disability.

Storage

Store in accordance with manufacturers instructions, see packaging.

End of Shelf Life

Storage Temperature Range

Storage Maximum Relative Humidity

Note: Average conditions may exceed 25°C/80%RH for limited periods. They can reach an average of 38°C/85%RH, provided that this is for no more than 3 months in the shelf life of the product.

When transporting this product use original packaging.

Approvals

This product has been shown to meet the Basic Safety Requirements under art. 10 and 11B of the European Community Directive 89/686 and are thus CE marked. The product was examined at the design stage by Berufsgenossenschaftliches Institut für Arbeitssicherheit, St. Augustin, Germany (0121).

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